
nosiboo^{*}pro2

electric nasal aspirator

reddot award 2014
winner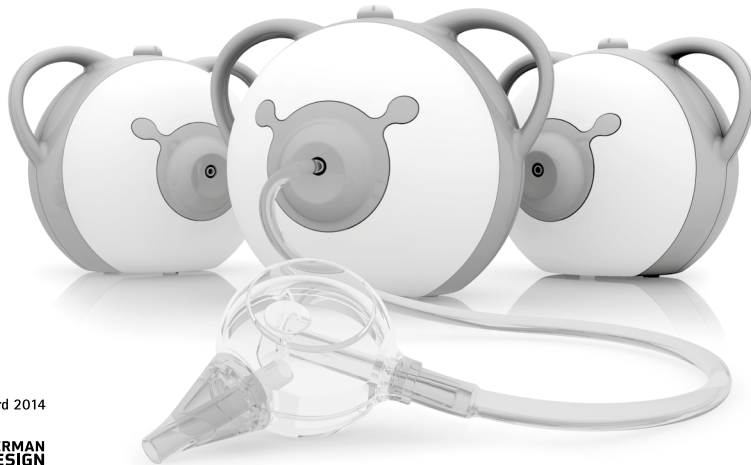

GERMAN
DESIGN
AWARD
SPECIAL
2016

Safety instructions

Table of contents

ENG	Safety instructions				
ARA	تعليمات السلامة	HIN	सुरक्षा के निर्देश	POR	Instruções de segurança
BUL	Инструкции за безопасност	HRV	Sigurnosne upute	RUM	Instrucțiuni de siguranță
CZE	Bezpečnostní pokyny	HUN	Biztonsági utasítások	RUS	Правила техники безопасности
DAN	Sikkerhedsanvisninger	ITA	Istruzioni di sicurezza	SLO	Bezpečnostné pokyny
DUT	Veiligheidsinstructies	JPN	安全上の注意	SLV	Varnostna navodila
EST	Ohutusjuhised	KOR	안전 지침	SPA	Instrucciones de seguridad
FIN	Turvallisuusohjeet	LAV	Norādījumi par drošību	SWE	Säkerhetsinstruktioner
FRA	Consignes de sécurité	LIT	Saugumo nurodymai	TUR	Güvenlik Talimatları
GER	Sicherheitshinweise	NOR	Sikkerhetsinstruksjoner	VIE	Chỉ dẫn an toàn
GRE	Οδηγίες ασφαλείας	POL	Instrukcje dotyczące bezpieczeństwa	ZHO	安全指示

nosiboo^{*}pro2[®]

electric nasal aspirator

ENG

Safety instructions

Dear Customer,

thank you for purchasing our Nosiboo Pro2 Electric Nasal Aspirator. Please, follow the instructions provided for your device. To obtain the very best performance for your nasal aspirator, please read the **Quick Guide** too. For more information we recommend reading the **User Manual**:

Attract Kft. is an ISO 13485 certified medical device manufacturer. This product is an MDD Legacy Device.

Actor EUDAMED No.: HU-MF-000004252

Please keep these documents for future reference.

Please read all instructions carefully before using the device.

The warranty will be only validated if the device was used according to its intended purpose and does not bear signs of damage.

Nosiboo Pro2 is a hygienic medical device. You may only exercise your right of withdrawal if the device is undamaged and the protective foil on the device and the nose tip is intact. For hygienic reasons, we are unable to refund the product if the protective foil has been opened.

Safety instructions

Type BF applied parts
(increased protection against electric shock)

Double insulation

Class IIa medical device

Name and address of the manufacturer

Read the user manual

For home use only

GTIN – Global Trade Item Number (UDI-DI)

Serial number (Unique device identifier)

Item number (type number)

Lot number (production identifier)

Keep dry

The housing protects against solid objects over 12.5 mm (4.92 Inches) entering the device; however, it does not protect against the infiltration of liquids.

Hazardous waste

General use

Use only as directed. Misuse can be dangerous.

The device should only be used by adults.

Do not use the nasal aspirator for nasal washing!

In case of unexpected failure or malfunction, immediately switch off the device.

Do not allow children to play with the device, it is NOT A TOY. (E.g., the power cable can wind easily around a child's neck.)

Do not leave the device unattended near children.

Do not use the device in the bathroom or in any other humid or wet environment.

Do not handle the power cable and the metal parts of the device with wet hands. Do not cover the vents on the back of the device while in use.

The device is intended to be used only for removing nasal mucus from a child's nose. It shall not be used for any other purposes.

Keep the device and its accessories out of reach of children.

Make sure that all parts are properly attached before using the device.

Do not use the device without the air mesh.

Make sure that the housing, the connecting tube, and the Colibri head are not damaged or cracked.

Use the device only with the Colibri head supplied by the manufacturer.

The part of the Colibri head attached to the connecting tube shall point downwards while in use.

Do not turn the Colibri head upside down meanwhile in use.

Cleaning, maintenance and storage

Keep the device away from heat (e.g. furnace, heater, or convector) because it can cause damage to the insulation.

Do not tie knots on the power cable or the connecting tube.

To avoid damage, do not place heavy items on the cable.

A damaged power cable may cause electric shock.

Do not use the power cable and the device if they are damaged or cracked. Do not remove the mounting screws from the housing. Do not reconstruct the device for other purposes. It may be life-threatening.

When you disassemble the Colibri head for cleaning, do not leave the parts within reach of children. Children may chew or swallow the small parts.

Do not wash the housing or the power cable with water or any other liquid. Do not handle the device with wet hands. This may cause electric shock or malfunction.

Always unplug the power cable from the outlet after using the device.

The device should be stored in a cool, dry place. Avoid direct sunlight or heat.

Power source

Use the device only with the power cable supplied by the manufacturer. Failure to do so may result in fire hazard.

Be sure the plug of the power cable is compatible with your power outlet!

The device should be connected to or disconnected from the power network only if the control button is in OFF position (the device is switched off).

Important information about the device

The Nosiboo Pro2 devices are supplied with an internal power supply generating extra-low voltage.

Disassembling the device housing is forbidden and life-threatening.

Specifications

Max. motor performance: 60 W

Adjustable motor performance: 10-60W

Differential pressure: 0.04-0.08 bar

Air flow: 21-42 l/min

Power supply: 100-240 V 50/60 Hz

IP class: IP21

Protection class: II

Type of applied parts: BF

Applied parts: the complete housing

Operation type: no continuous operation

Operation length: recommended between 0.5-1.5 minutes (The device switches off automatically after 15 minutes of use.)

Thermal protection activation: 80°C

Average housing temperature: 30-35°C after 1 min of continuous use

Noise level: 52-62 dB

Operating temperature range: from -10°C to +40°C

Operating relative humidity: max. 85%

Storage and transportation temperature: from -10°C to +85°C

Storage and transportation relative humidity: 10-65%

Expected lifetime of the device housing (electrical parts): 3 years

Expected lifetime of the Colibri head: 3 months

Weight (housing): 750 g

Dimensions (housing, W x H x D): 200 x 155 x 175 mm

Dimensions (complete package): 180 x 180 x 225 mm

MD Category: II.a

NBOG: MD1102

MDD/MDR: MDD Legacy Device

Actor EUDAMED No.: HU-MF-000004252

We reserve the right to make technical changes.

The manufacturer shall not be held responsible for the failure of products from unauthorised import sources!

Contact

Manufacturer contact details.

Attract Kft.

H-7622 Pécs, Siklósi út 1/1., Hungary

Tel.: +36 72 551 642 (Mo-Fr: 8 am – 4 pm)

E-mail: customerservice@nosiboo.com

PRRC (Person Responsible for Regulatory Compliance):

Attract Kft. – Quality Manager

H-7622 Pécs, Siklósi út 1/1., Hungary

Tel.: +36 72 551 642 (Mo-Fr: 8 am – 4 pm)

E-mail: quality@nosiboo.com

For more information we suggest reading the user manual and our quick guide as well.

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