# nosiboo<sup>\*</sup>pro<sup>®</sup>

electric nasal aspirator

## User Manual

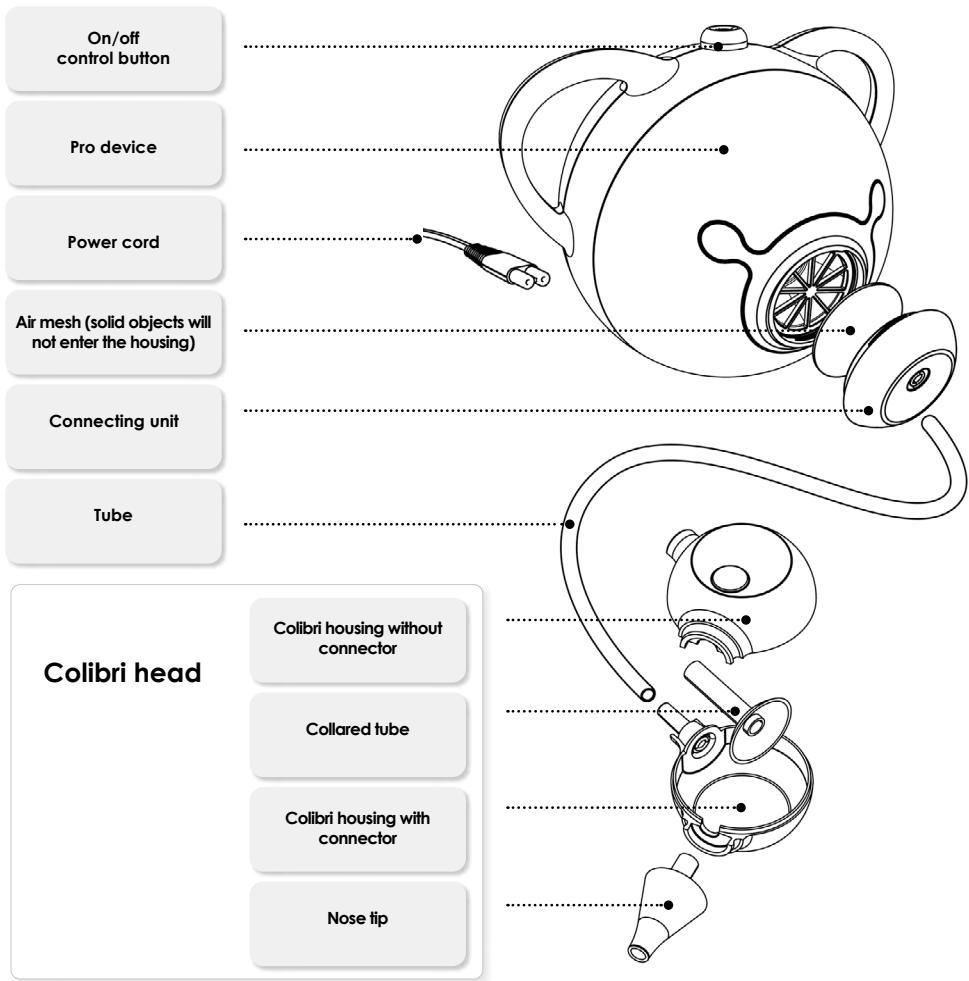
reddot award 2014  
winner

- |    |    |    |    |    |    |    |
|----|----|----|----|----|----|----|
| EN | AR | BG | CS | DA | DE | EL |
| ES | ET | FI | FR | HR | HU | IT |
| LT | LV | NL | NO | PL | PT | RO |
| SK | SL | SV | TR | VI | ZH |    |

# Table of contents

On/off control button

Pro device

Power cord

Air mesh (solid objects will not enter the housing)

Connecting unit

Tube

**Colibri head**

Colibri housing without connector

Collared tube

Colibri housing with connector

Nose tip

# USER MANUAL

## Foreword

Dear Customer,  
thank you for purchasing our KSCI, PTPA, Red Dot Design and German Design award-winning Nosiboo Pro nasal aspirator.

Please, follow the instructions provided in this User Manual to obtain the very best performance from your nasal aspirator.

## Table of contents

1. Package contents
2. Safety instructions
3. Use
4. Before use
5. Instructions for use
6. Cleaning
7. Storage and maintenance
8. Troubleshooting
9. Important information about electromagnetic interference
10. Environmental impact
11. Specifications
12. Accessories, spare parts

*Read all instructions before use.*

CE 2409

**The Nosiboo Pro Nasal Aspirator is a medical device.** You may only return the device for a refund if the device is unused and undamaged, and the protective foil on the Colibri head and on the nose tip are intact.

## 1. Package contents

Nosiboo Pro

Tube

Colibri head +  
Newborn's nose tip

Standard nose tip

Power cord

Cleaning brush

ATTRACT Kft. is an ISO 13485 certified medical device manufacturer. This product is an MDD Legacy Device.

Actor EUDAMED No.: HU-MF-000004252

## 2. Safety instructions

### Safety signs

Type BF applied parts (increased protection against electric shock)

For home use only

Keep dry

Double insulation

Item number (type number)

Warning!

Class IIa medical device

Serial number (Unique device identifier)

Caution!

Name and address of the manufacturer

GTIN - Global Trade Item Number (UDI-DI)

DO

Read the Safety instructions

Lot number (production identifier)

DON'T

Read the online User manual

Protection against solid particles of diameter greater than 12,5 mm; protected from condensation.

Hazardous waste

### General use

**Use only as directed. Misuse can cause injury or death.**

**The device should only be operated by adults.**

Do not allow children to play with the device. It is **NOT A TOY**. (The power cord can easily wind around a child's neck.)

Do not leave the device unattended near children.

Do not use the device in the bathroom or in any other humid or wet environment.

Do not handle the power cord with wet hands.

Do not cover the vents on the back of the device while in use. Hot air exits the device through the vents.

Do not use the nasal aspirator for nasal washing!

**The device should be on a flat, stable surface during use.**

**The connection point of the Colibri head and the tube should point downwards while in use.**

**Always switch off the device after use by turning the control button. Do not switch off the device by unplugging the power cord!**

**Always thoroughly clean the Colibri head after use (see the "Cleaning" section).**

**Read and follow the instructions in the "Important information about electromagnetic interference" section.**

**Read and follow the instructions in the "Environmental impact" section if you would like to dispose of the device or any part of it.**

The device is intended to be used only for removing nasal mucus from a child's nose.

Please consult your healthcare provider about when and how often you should clear your baby's nose, or if redness occurs.

Keep out of reach of children.

Make sure that all parts are properly attached before using the device.

Make sure that the housing, the tube, and the Colibri head are not damaged or cracked.

Stop using the device if mucus or saline solution has been drawn into the tube. Disconnect it from the electrical outlet, empty the Colibri head, clean and dry the Colibri head and the tube, check the air mesh and replace it if it is wet or clogged.

Double-check that you have inserted the collared tube into the Colibri head after cleaning.

Do not insert the nose tip into the baby's mouth or ears.

Do not force or insert the nose tip too far into the nose.

Do not press the nose tip against the inner side of the nostril.

Do not use the device if nasal sores, inflammation, or nosebleed are present. If pain or bleeding occurs, stop use immediately and consult your doctor.

Do not use the device if you see that the tube or the Colibri head are damaged. Replace the damaged part. (A list of spare parts can be found in the "Accessories, spare parts" section.)

Do not use the Colibri head and the device without the collared tube.

Do not use more than one teaspoon of saline solution. Excess solution can enter the connecting tube and can cause damage to the device.

The device cannot be used in oxygen-rich environments and in clinical circumstances! The medical device is for home use only.

Do not attach the Colibri head to other devices.

Do not unplug the power cord from the electrical outlet until the device is completely shut down.

## Cleaning, Maintenance, and Storage

**Protect the power cord against damage.**

**Using a damaged power cord can cause short circuit (short-circuit fire).**

**Keep the power cord away from heat (e.g., furnace, heater, or convector) because it can cause damage to the insulation.**

**Turn the device off and grasp the plug to unplug the power cord from the electrical outlet when assembling, disassembling, and cleaning the device.**

**Do not break the power cord, and do not place heavy items on the cord, to avoid damage. Grasp the plug to unplug the power cord from the electrical socket. A damaged cord may cause electric shock.**

**Do not remove the mounting screws from the housing. Do not reconstruct the device for other purposes.**

**A broken, damaged, worn, frayed or short-circuited power cord can cause an electrical fire, so do not use it!**

**Do not leave the parts of the dismantled Colibri head within reach of children. Children may swallow the small parts.**

**Do not wash the device housing or the power cord in water or any other liquid. Do not handle the power cord with wet hands. This may cause electric shock or malfunction.**

**Make sure that all parts are dried thoroughly before the next use.**

**Double-check that you have inserted the collared tube into the Colibri head after cleaning.**

**When not in use, always unplug the device from the outlet by grasping the plug.**

**To prevent damage, the connecting unit should not be exposed to heat.**

**It is not allowed to use a dishwasher, boiling water, microwave oven, vaporizer, or baby bottle sterilizer for cleaning the spare parts, because they might deform by heat!**

**Store in a cool, dry place. Avoid direct sunlight or heat.**

## Power Source

**Protect the power cord against damage.**

**Turn the device off and grasp the plug to unplug the power cord from the electrical outlet when assembling, disassembling, and cleaning the device.**

**In case of unexpected failure or malfunction, immediately switch off the device and unplug the power cord from the electrical outlet.**

**Be sure to use the device only with a 230 V 50 Hz AC.**

**Do not use the device if there is a breakage or crack in the housing or power cord.**

**Do not break the power cord, and do not place heavy items on the cord, to avoid damage. Grasp the plug to unplug the power cord from the electrical socket. A damaged cord may cause electric shock.**

**When not in use, always unplug the device from the outlet by grasping the plug.**

**Do not unplug the power cord from the electrical outlet until the device is completely shut down.**

## 3. Use

The Nosiboo Pro nasal aspirator is intended for removing nasal mucus from a child's nose.

The Colibri head has a patented design that prevents mucus from entering the device, simplifying the cleaning of the Colibri head, making the daily use of the nasal aspirator easier.

### Intended purpose:

- \* to ensure clearer nasal breathing for infants,
- \* to relieve symptoms of common childhood colds by aspirating nasal mucus,
- \* to relieve childhood allergy-like symptoms by removing nasal mucus.

### Do not use:

- \* if the child has haemophilia,
- \* if the child has frequent nosebleeds,
- \* if the child is taking blood-thinning medicines,
- \* in the event of congenital malformations (e.g. cleft palate).

**The device should only be operated by adults.**

**The device is intended to be used only for removing nasal mucus from a child's nose.**

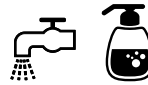
**The device is not a nasal irrigator! Do not use large amounts of liquid during nasal aspiration! The Colibri head can hold up to a teaspoon (5 ml) of thin liquid.**


## 4. Before use

1. Disassemble the Colibri head and clean and dry its parts.

Parts of the Colibri head:

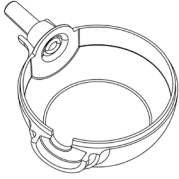
Colibri housing with  
connector

Colibri housing without  
connector



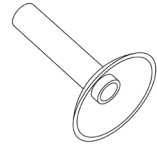
Nose tip

Collared  
tube

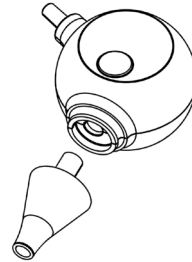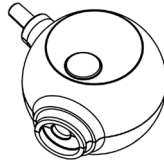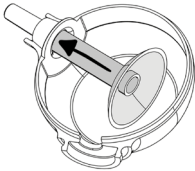
2. Assemble the Colibri head.

a) Insert the collared tube  
into the side of the  
Colibri housing which is  
supplied with a  
connector.

b) Place the Colibri  
housing that has no  
connector in the  
correct position.

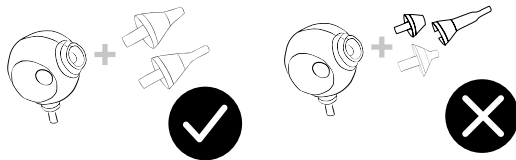
c) Push the nose tip into  
place.


**Use original Nosiboo parts!**  
Our nasal aspirators work effectively only with original Nosiboo parts/accessories.

### 3. Attach the Colibri head to the device.

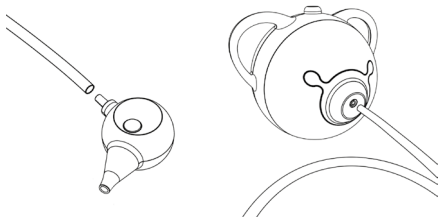- Attach one end of the connecting tube to the Colibri head.
- Attach the free end of the connecting tube to the device housing.

### 4. Attach one end of the power cord to the device.

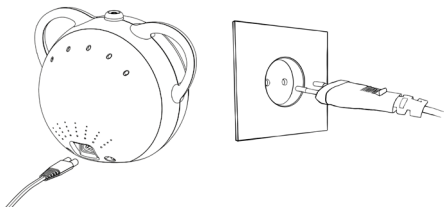Plug the other end of the power cord into a 230 V 50 Hz AC outlet.

### 5. The device is ready for use.

The device should be connected to or disconnected from the power network only if the control button is in OFF position (the device is switched off).

Be sure to use the device only with a 230 V 50 Hz AC. Failure to do so may cause electric shock or fire.

Clean the parts of the Colibri head before every use.

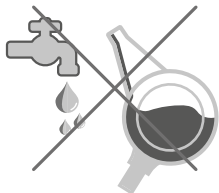
Make sure that all parts are clean and dried thoroughly before use.


Do not use the Colibri head and the device without the collared tube.

## 5. Instructions for use

**Do not aspirate water with the device!**

**Do not use the nasal aspirator for nasal washing!**

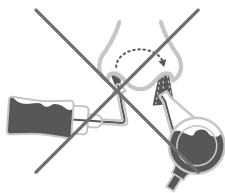Aspirating water with the electric device will result in loss of warranty.

**1. The device is ready for use.**

**2. Switch on the device by turning the control button.**

The device should be connected to or disconnected from the power network only if the control button is in OFF position (the device is switched off).

**NOTE:** Any time you are not actively using the device, switch it off. After the motor cycles off completely, you can restart it.

**3. Select the correct suction power.**

low setting for newborns

medium setting for babies

high setting for small children and for preschoolers

**TIP:** If the mucus is thin, you can use the device at a lower setting. If it is thick, you can use the device at a higher setting than normal.

**4. Hold your child in a stable position.**

Sit your child onto your lap, hold them in your arms or sit them into an upright position supporting their back with a pillow. You may also sit them in an infant bouncer or infant seat.

**5. Insert the clean, dry nose tip into the child's nostril.**

In case of newborns, it is sufficient to hold the nose tip at the opening of the nostril. The connection point of the connecting tube should point downwards while in use.

**NOTE:** In case of swollen nasal mucosa or blocked nasal openings, use decongestant solutions or nasal sprays 30 minutes before application.

**6. Move the nose tip around gently to make sure that you get all areas of the nostril.**

**NOTE:** Do not pinch or block the other nostril while using the nasal aspirator.

**Newborn's nose tip:**  
0-6 months (baby's age)

**Standard nose tip:**  
6 months and older

The suggested duration of contact aspiration is 10 seconds per nostril.

Dry, very thick mucus can be dissolved by a mild saline solution. After using the solution, wait a few minutes, wipe excess discharge with a tissue, then reuse the nasal aspirator as directed.

To avoid damage to the device, do not use more than a teaspoon of solution at a time. The Colibri head can hold up to a teaspoon (5 ml) of thin liquid.

This device is not a nasal irrigator.

## 7. Switch off the device after use with the control button.

The device should be connected to or disconnected from the power network only if the control button is in OFF position (the device is switched off).

NOTE: Always switch off the device after use.

TIP: The Nosiboo Pro Nasal Aspirator can be used several times a day. It is recommended that you clear the airways before bedtime and feeding to ensure the child breathes peacefully.

NOTE: Nose drops and other preparations should be used as directed by your healthcare professional.

The device should only be operated by adults.

Do not allow children to play with the device. It is NOT A TOY.

Do not cover the vents on the back of the device while in use. Hot air exits the device through the vents.

Do not use the device in the bathroom or in any other humid or wet environment.

Do not handle the power cord with wet hands.

Do not break the power cord during use and storage.

Make sure that all parts are clean and dried thoroughly before use.

Always switch off the device after use by turning the control button. Do not switch off the device by unplugging the power cord!

The device should be on a flat, stable surface during use.

Do not unplug the power cord from the electrical outlet until the device is completely shut down.

Do not use the Colibri head and the device without the collared tube.

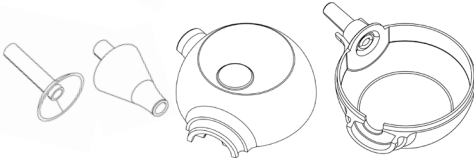
Do not cover the device, and do not use it on a soft surface which could block the ventilation holes (couch, pillow, bedclothes etc.)

## 6. Cleaning


1. Remove the connecting tube and the nose tip, then take apart the Colibri head.


2. Wash the collared tube, the nose tip, and the Colibri housing in lukewarm water and dishwashing detergent and let them dry.

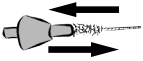
Use the brush for cleaning the nose tip.

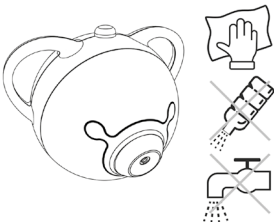3. Clean the housing with a dry, soft cloth; do not use any liquids, detergents, or chemicals.

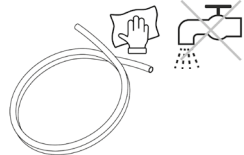4. Wipe the connecting tube with a cloth if necessary.

**Turn the device off and grasp the plug to unplug the power cord from the electrical outlet when assembling, disassembling, and cleaning the device.**

**Do not wash the device housing or the power cord in water or any other liquid. Do not handle the power cord with wet hands. This may cause electric shock or malfunction.**

**Do not wash the connecting tube.**

**Thoroughly clean the Colibri head after use.**

**Make sure that all parts are clean and dried thoroughly before use.**

**If liquid gets into the connecting tube due to improper use, make sure that it is completely dry before using it again.**

**Do not remove the mounting screws from the housing. Do not reconstruct the device for other purposes.**

**To avoid damage, make sure the device is not exposed to heat.**

**It is not allowed to use a dishwasher, boiling water, microwave oven, vaporizer, or baby bottle sterilizer for cleaning the spare parts, because they might deform by heat!**

**Do not use any liquids, detergents or chemicals for cleaning the device housing and the connecting tube.**

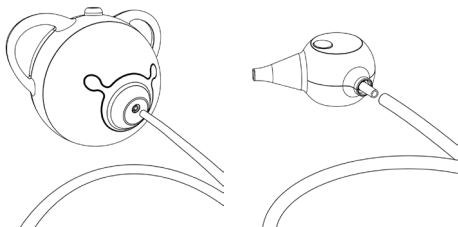
## 7. Storage and maintenance

### Storage

1. Unplug the power cord from the wall socket only after the device has completely shut off.

**NOTE:** After the device is switched off, the power cord should be unplugged from the electrical outlet and from the device by grasping the plug.

2. Disconnect the connecting tube from the device and the Colibri head.

3. Keep the device in an enclosed space where it is protected from dust and damage (e.g., in a cabinet).

**NOTE:** Do not store near a heat source.

### Maintenance

There is a special air mesh behind the connecting unit which prevents solid objects and dust from entering the housing. In the event of an unexpected drop in performance, it is recommended that you check the air mesh. If the air mesh is dirty or clogged, replace it with a spare air mesh. (A list of spare parts can be found in the "Accessories, spare parts" section.)

**Protect the power cord against damage.**

**Keep the power cord away from heat.**

**Do not leave the parts of the dismantled Colibri head within reach of children. Children may swallow the small parts.**

**Do not break the power cord, and do not place heavy items on the cord, to avoid damage. Grasp the plug to unplug the power cord from the electrical socket. A damaged cord may cause electric shock.**

**Do not wind the power cord around the device.**

**A damaged power cord can cause electric fire!**

**Make sure that all parts are clean and dried thoroughly before use.**

**Store the device in a clean, dry place. Avoid direct sunlight or heat.**

## 8. Troubleshooting

<b>Malfunction</b>	<b>Reason</b>	<b>Recommendation</b>
The device cannot be switched on.	The device is not connected to the power network.	Connect the power cord to the device, then to the electric socket.
The suction intensity is decreased.	The nose tip is blocked or the Colibri head was incorrectly assembled.	Take the Colibri head apart and clean it. After drying the parts of the Colibri head, assemble them according to the User Manual.
The suction intensity is decreased.	The air mesh is clogged.	Replace the air mesh according to the User Manual's "Storage and maintenance" section.
Liquid or mucus got into the connecting tube.	The Colibri head was not held in the correct position.	Switch off the device and clean and dry the Colibri head and the connecting tube. Check the air mesh and replace it if necessary.
Liquid or mucus got into the connecting tube.	The connection point of the tube is not pointed downwards.	Switch off the device and clean and dry the Colibri head and the connecting tube. Check the air mesh and replace it if necessary.
Liquid or mucus got into the connecting tube.	Too much saline solution was used.	Reduce the volume of the saline solution to less than a teaspoon (5 ml). Switch off the device and clean it, then dry the Colibri head and the connecting tube. Check the air mesh and replace it if necessary.

## 9. Important information about electromagnetic interference

Home wireless network equipment, mobile phones, and wireless phones may affect the operation of certain medical equipment, as electromagnetic interference can occur between devices. Electromagnetic interference may cause harmful malfunction in the operation of certain medical devices, potentially causing emergency situations. It is recommended to keep at least 5 m distance between the devices!

Due to the abovementioned reasons, medical devices must not interfere with any other equipment. Do not use near high-performance radiation-emitting devices.

The Nosiboo Pro Nasal Aspirator is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled.

The Nosiboo Pro nasal aspirator is certified as compliant with the requirements of IEC 60601-1-2:2014; EN 6061-1-2:2018 (EMC test); EN 6061-1-11:2015; EN 6061-1:2017 standards.

## 10. Environmental impact

Dispose of the device in accordance with your local recycling and waste disposal practices and requirements.

The Colibri head does not present a risk to the environment and can be disposed of as household waste.

## 11. Specifications

**Max. motor performance:** 500 W

**Adjustable motor performance:** 150-390W

**Performance:** 420 VA

**Differential pressure:** 0.04-0.08 bar

**Max. air flow:** 30-42 l/min

**Power supply:** 230 V 50 Hz

**IP class:** IP21

**Protection class:** II

**Type of applied parts:** BF

**Applied parts:** the complete housing

**Operation type:** no continuous operation

**Operation length:** on 15 min/appr. off 50 min

**Duration of continuous use:** max 15 min.

**Safety switch release time:** after 15 minutes

**Tripping temperature of the thermal switch:** 95 °C

**Average housing temperature:** 30-35 °C after 1 min continuous use

**Noise level:** 60-70 dB

**Expected life of the housing (electrical parts):** 3 years

**Expected life of the Colibri head:** 3 months

**Diameter of the nasal insertion end of the standard nose tip:**

max. 7.0 mm

**Length of the nasal insertion end of the standard nose tip:**

max. 8.0 mm

**Diameter of the nasal insertion end of the newborn's nose tip:**

max. 4.9 mm

**Length of the nasal insertion end of the newborn's nose tip:**

max. 12.6 mm

**Weight:** 1090 g

**Dimensions (housing, W x H x D):** 198 x 162 x 172 mm

**Dimensions (complete package):** 180 x 180 x 225 mm

**Operating conditions:**

1. **Temperature range:** -5 °C to +40 °C

2. **Relative humidity:** max. 75%

3. **Atmospheric pressure / barometric pressure range:** 525-795

mmHg / 70-106 kPa

**Transportation conditions:**

1. **Temperature range:** -20 °C to +65 °C

2. **Relative humidity:** 10-75%

3. **Atmospheric pressure / barometric pressure range:** 150-795

mmHg / 20-106 kPa

**Storage conditions:**

1. **Temperature range:** -20 °C to +65 °C

2. **Relative humidity:** 10-75%

3. **Atmospheric pressure / barometric pressure range:** 525-795

mmHg / 70-106 kPa

**MD Category:** II.a

**NBOG:** MD1102

**MDD/MDR:** MDD Legacy Device

**Actor EUDAMED No.:** HU-MF-000004252

We reserve the right to make technical changes.

## 12. Accessories, spare parts

In the European Union, parts can be ordered from the specialist service station:

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**Tel.: +36 72 551 642 (Mo-Fr: 8 am – 4 pm)**

**E-mail: customerservice@nosiboo.com**

In countries outside the European Union, spare parts are available at the distributors of Nosiboo products.

Power cord EU  
300026

Power cord UK  
300054

Connecting unit  
300007 \* blue  
300008 \* pink  
300009 \* green  
300076 \* grey  
300077 \* sage

For information on the warranty, please see the link below:  
<https://en.nosiboo.eu/downloads/>

The manufacturer shall not be held responsible for the failure of products from unauthorised import sources!

## Nosiboo Pro Accessory Set:

Accessory pack consisting of Nosiboo Pro/Pro2 spare parts. All the parts included in the package can be used for both types of the device.

**Available at the manufacturer's webshop (<https://shop.nosiboo.eu/>), at Amazon, or the distributors of Nosiboo products.**

ACS-01-01 blue

- \* 1 Colibri head  
+ newborn's nose tip
- \* 1 standard nose tip
- \* 1 tube
- \* 1 air mesh
- \* 1 cleaning brush

ACS-01-04 grey

- \* 1 Colibri head  
+ newborn's nose tip
- \* 1 standard nose tip
- \* 1 tube
- \* 1 air mesh
- \* 1 cleaning brush

ACS-01-02 pink

- \* 1 Colibri head  
+ newborn's nose tip
- \* 1 standard nose tip
- \* 1 tube
- \* 1 air mesh
- \* 1 cleaning brush

ACS-01-13 sage

- \* 1 Colibri head  
+ newborn's nose tip
- \* 1 standard nose tip
- \* 1 tube
- \* 1 air mesh
- \* 1 cleaning brush

ACS-01-03 green

- \* 1 Colibri head  
+ newborn's nose tip
- \* 1 standard nose tip
- \* 1 tube
- \* 1 air mesh
- \* 1 cleaning brush

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